

# F1000 Keyboard Reference Chart

alt Q Quit  
 alt N Start New Session  
 alt 1, 2, 3 Select Screen  
 Spacebar, 1,2,3,4, Mark Event

alt S Save & Continue  
 ctrl S Save & Restart  
 alt E Select EMG Save

F12 Info Display off/on  
 F11 Temperature 1 Autozero  
 ctrl F11 Temperature 2 Autozero  
 alt F11 EDR Autozero

| EMG Filter                        |                   |                 |                 |                 |                  |                            |                    |                             |
|-----------------------------------|-------------------|-----------------|-----------------|-----------------|------------------|----------------------------|--------------------|-----------------------------|
| Shft<br>Ctrl F1                   | Shft<br>Ctrl F2   | Shft<br>Ctrl F3 | Shft<br>Ctrl F4 | Shft<br>Ctrl F5 | Shft<br>Ctrl F6  | Shft<br>Ctrl F7            | Shft<br>Ctrl F8    | Ctrl F9                     |
| 25<br>100                         | 25<br>500         | 50<br>500       | 100<br>500      | 200<br>600      | 400<br>800       |                            |                    |                             |
| EMG Smoothing                     |                   |                 |                 |                 |                  |                            |                    |                             |
| Shft<br>Alt F1                    | Shft<br>Alt F2    | Shft<br>Alt F3  | Shft<br>Alt F4  | Shft<br>Alt F5  | Shft<br>Alt F6   | Shft<br>Alt F7             | Shft<br>Alt F8     | Shft<br>Alt F9              |
| 5                                 | 10                | 20              | 40              | Slow<br>80      |                  |                            |                    |                             |
| Temperature (Use Ctrl for Chan 2) |                   |                 |                 |                 |                  |                            |                    |                             |
| F1                                | F2                | F3              | F4              | F5              | F6               | F7                         | F8                 | F9                          |
| CENTER<br>                        | Range<br>Down<br> | Range<br>Up<br> | OFF<br>ON<br>   | <br>OFF<br>ON   | BBB<br>OFF<br>ON | Sound<br>Normal<br>Reverse | SOUND<br>OFF<br>ON | MASTER<br>TEMP<br>OFF<br>ON |
| EDR (Use alt)                     |                   |                 |                 |                 |                  |                            |                    |                             |
| alt F1                            | alt F2            | alt F3          | alt F4          | alt F5          | alt F6           | alt F7                     | alt F8             | alt F9                      |
| Ref<br>                           | Range<br>Down<br> | Range<br>Up<br> | x<br>           | +/-<br>         | BBB<br>OFF<br>ON |                            | SOUND<br>OFF<br>ON | EDR<br>OFF<br>ON            |

## Numeric Keypad

| EMG CONTROLS       |                     |                        |                        |
|--------------------|---------------------|------------------------|------------------------|
| Use Alt for ENG 2  |                     |                        |                        |
| 7                  | 8                   | 9                      | -                      |
|                    | ↑<br>THRESH<br>UP   | STOP<br>THRESH<br>UP   | Sound<br>Level<br>Down |
| 4                  | 5                   | 6                      | +                      |
| SCALE<br>DOWN<br>← | SOUND<br>OFF<br>ON  | SCALE<br>UP<br>→       | Sound<br>Level<br>Up   |
| 1                  | 2                   | 3                      |                        |
|                    | ↓<br>THRESH<br>DOWN | STOP<br>THRESH<br>DOWN |                        |
| Ins                |                     | Del                    |                        |
|                    |                     | OFF<br>ON              |                        |